

How to start a NeurOptimal® Session

1

Identify items.

Paste



zAmp & USB cable



Tablet/Computer



2

Launch NeurOptimal®.

Launch NeurOptimal® by tapping on the NeurOptimal® icon in the Task Bar at the bottom of the screen



Vault



Media Player




3

Connect USB cable to zAmp and Computer or Tablet.



4


Apply Sensors.

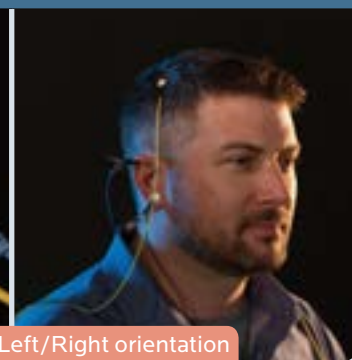

Note color coding and Left/Right orientation


5

Double tap "Client" to add your name (if needed).

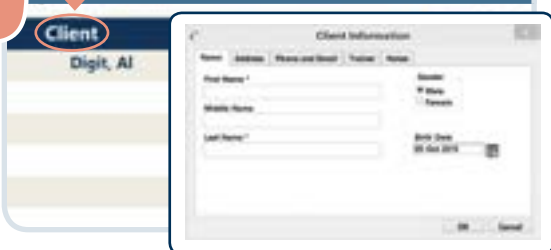

6

Double tap your name to start a session.

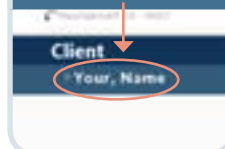

IMPORTANT

Please always have your system connected to the internet.

Please contact your rental provider at [719-431-0511](tel:719-431-0511) with any questions.